

# YOU CAN DO IT REMOTELY

# LibreView

## WITH YOUR FREESTYLE LIBRE READER

If you scan with the FreeStyle Libre reader, you can create a LibreView account, to share your data.



- Go to [www.LibreView.com](http://www.LibreView.com)
- Click on **Sign up**
- Choose the **LibreView account for patients** and follow on-screen instructions
- **Connect** your FreeStyle Libre reader
- Press **Upload data** and follow on-screen instructions



### DID YOU KNOW?

You can scan with the FreeStyle LibreLink<sup>1</sup> app on your phone, and easily share your glucose data with your healthcare professional's practice through LibreView<sup>2</sup>



**1800 801 478**

If you have any questions or require support with the FreeStyle Libre system, please call FreeStyle Libre Customer Care or visit [FreeStyleLibre.com.au](http://FreeStyleLibre.com.au)



*life. to the fullest.*

**Abbott**

THIS PRODUCT MAY NOT BE RIGHT FOR YOU. READ THE WARNINGS BEFORE PURCHASE AND FOLLOW THE INSTRUCTIONS. Find out more at [FreeStyleLibre.com.au](http://FreeStyleLibre.com.au)

This product is indicated for measuring interstitial fluid glucose levels in people (age 4 and older) with insulin-requiring diabetes.

†Scanning the sensor to obtain glucose values does not require lancets. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycaemia or impending hypoglycaemia is reported by the System or when symptoms do not match the System readings.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC.

1. The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. 2. LibreView minimum system requirements: Safari release 10.1, Internet Explorer 11, Firefox release 32.0, or Chrome release 37.0, installed on a computer running Mac OS X Yosemite (10.10), Windows 7 SP1, Windows 8.1, or Windows 10 Anniversary update. LibreView Device Drivers are installed in order to upload data from a device connected to the computer. Refer to the LibreView Quick Start Guide for additional information.

# There are two easy ways to share your data with your healthcare professional


# LibreView



## Option A:

- 1 Ask your healthcare professional to **invite you** to share your data
- 2 You will receive an email inviting you to share. Click **Accept Invitation** and you will be directed to the **LibreView** website
- 3 Log into your **LibreView** account or create a LibreView account.

## Option B:

- 1 Ask your healthcare professional for his/her **practice ID**
- 2 Log into LibreView or create a LibreView account. Then click on the  icon, at the top right of the screen and select **Account Settings**
- 3 Click on **My Practices**
- 4 Enter your healthcare professional's Practice ID



**1800 801 478**

If you have any questions or require support with the FreeStyle Libre system, please call FreeStyle Libre Customer Care or visit [FreeStyleLibre.com.au](http://FreeStyleLibre.com.au)



*life. to the fullest.*

**Abbott**

THIS PRODUCT MAY NOT BE RIGHT FOR YOU. READ THE WARNINGS BEFORE PURCHASE AND FOLLOW THE INSTRUCTIONS. Find out more at [FreeStyleLibre.com.au](http://FreeStyleLibre.com.au)

This product is indicated for measuring interstitial fluid glucose levels in people (age 4 and older) with insulin-requiring diabetes.

†Scanning the sensor to obtain glucose values does not require lancets. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycaemia or impending hypoglycaemia is reported by the System or when symptoms do not match the System readings.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered

in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC.

11 The FreeStyle LibreLink app and the FreeStyle Libre reader have similar but not identical features. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycaemia or impending hypoglycaemia is reported by the FreeStyle LibreLink app. 12 The FreeStyle Libre sensor communicates with the FreeStyle Libre reader that started it or the FreeStyle LibreLink app that started it. A sensor started by the FreeStyle Libre reader will also communicate with the FreeStyle LibreLink app. 13 LibreLinkUp is a mobile application developed and provided by Newyu, Inc. Use of FreeStyle LibreLink

and LibreLinkUp requires registration with LibreView, a service of Abbott and Newyu Inc. FreeStyle Libre readers and sensors can be obtained from Abbott. 14 It is possible to accept the LibreLinkUp invitation and receive messages and warnings upon doing so or to reject the invitation. You should base your decision upon your knowledge and experiences in being able to react accordingly when obtaining a glucose value that is too high or too low. 15 Transfer of glucose data between applications depends on cellular connectivity. 16 LibreView data can be viewed in the Safari Browser on Mac OS X Mountain Lion or higher computers and on iOS 6 or higher mobile devices. Currently, uploading of glucose data is only supported on Windows-based computers. Minimum system requirements are Windows Vista with IE10 or the latest versions of Google Chrome and Mozilla Firefox, running on a 550MHz Pentium III, 512MB DRAM, 2G Hard Drive, USB 2.0, LCD Screen

with resolution of 1024X768. 17 LibreView is not intended to be a primary glucose monitor: home users must consult their primary device(s) and consult a healthcare professional before making any medical interpretation and therapy adjustments from the information in the software, and healthcare professionals should use information in the software in conjunction with other clinical information available to them.

FreeStyle, Libre, and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions. Information contained herein is for distribution outside of the USA only. Abbott Diabetes Care, 666 Doncaster Road, Doncaster, Victoria 3108, Australia. If you have any questions, please contact our Customer Service Team 1800 801 478. ABN 95 000 180 389 © 2020 Abbott ADC-21146 v1.0